

V.A.C. VERAFLU CLEANSE CHOICE™ Dressing System Application Summary

This Application Summary should be used only in conjunction with the safety and application information provided in the *Instructions for Use* provided in the V.A.C. VERAFLU CLEANSE CHOICE Dressing cartons.

1. Wound Preparation and Application of 3M™ Cavilon™ No Sting Barrier Film

- Ensure debridement of all necrotic, non-viable tissue, including bone, eschar, or hardened slough, as prescribed by treating physician.
- Perform thorough wound and periwound area cleaning per physician order or institution protocol prior to each dressing application.
- Skin should be clean & dry prior to application of 3M™ Cavilon™ No Sting Barrier Film
- Wipe uniform coating of film over the entire area of concern
- Allow to thoroughly dry before applying drape




2. Placement of Wound Contact Layer

- Protect sensitive structures, vessels, and organs as needed
- Assess wound dimensions and pathology, including the presence of undermining or tunnels
- Cut the Thin or Thick Cover layer of foam to fill explored tunnels and undermining. Be sure foam in tunnels makes contact with cover layer
- Cut Wound Contact Layer as needed to cover the entire wound bed and gently place in wound



3. Filling the Wound With The Cover Layer

- Cut Thick or Thin Cover Layer as needed to fill the wound depth and gently place in the wound on top of the wound Contact Layer

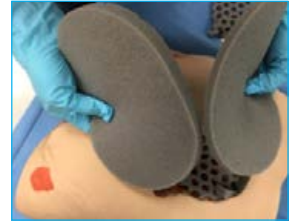

4. V.A.C.® Advanced Drape Application

- Trim the drape to cover the foam and at least an additional 3-5cm border of intact periwound tissue
- Remove Layer #1 to expose the adhesive and apply to cover layer and intact skin while ensuring skin wrinkles and folds are smoothed
- Remove Layer #2 and pat the drape to ensure an occlusive seal
- Remove the Ruler/Handling Bars



5. V.A.C. VERAT.R.A.C.™ Pad Placement

- Choose pad application site. Give particular consideration to tubing positioning to allow for optimal flow, and avoid placement over bony prominences or within creases in the tissue
- Cut an approximately 2.5cm hole through the drape to affix V.A.C. VERAT.R.A.C.™ Pad
- Remove both backing layers 1 and 2 to expose adhesive
- Position and apply V.A.C. VERAT.R.A.C.™ Pad over the 2.5cm hole
- Pull back on blue tab to remove pad stabilization layer




NOTE:

- Dressings should be changed every 48-72 hours, but no less than three times per week, with frequency adjusted by the clinician as appropriate
- Do not cut dressing over wound bed
- Do not pack foam into wound
- Do not use the wound contact layer with holes in tunnels or undermined areas

V.A.C. VERAFLU CLEANSE CHOICE™ Dressing: A Simple Solution For Complex Wounds

Thin Cover Layer
Thickness (0.8cm)

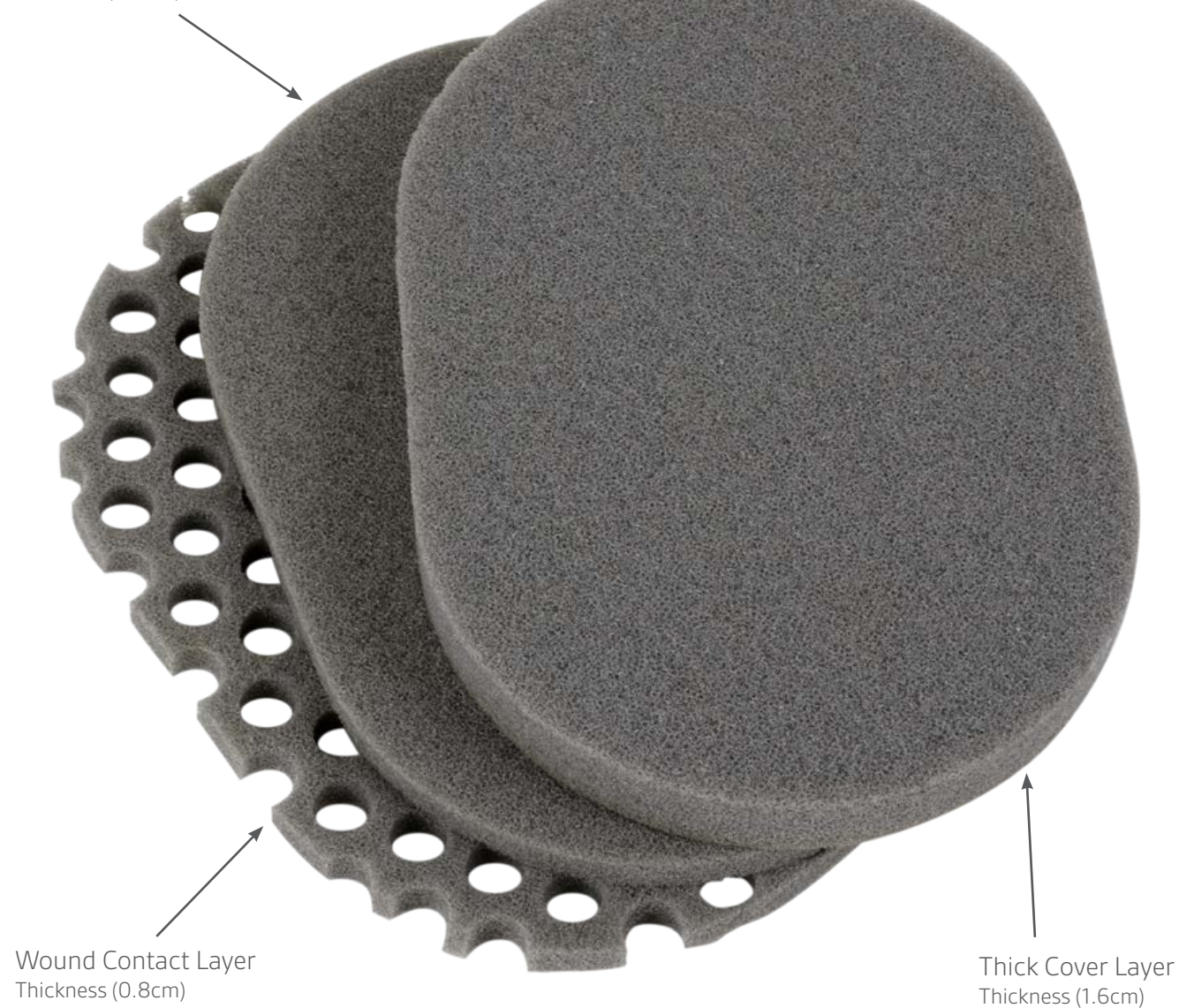

For more information, contact your [Acelity Representative](#)

Note: Specific indications, contraindications, warnings, precautions, and safety information exist for KCI products and therapies. Please consult a physician and product instructions for use prior to application. This material is intended for healthcare professionals only.

Copyright 2017 KCI Licensing, Inc. All rights reserved. 3M and Cavillon are trademarks of 3M Company. All other trademarks designated herein are proprietary to KCI Licensing, Inc., its affiliates and/or licensors. DSL#16-1279.OUS (1/17)