

SILVERCEL™ NON-ADHERENT DRESSING



What is it?

SILVERCEL™ NON-ADHERENT Dressing with Silver is a non-woven pad composed of alginate, carboxymethylcellulose (CMC) and silver coated nylon fibres, with a non-adherent wound contact layer.

What is it for?

SILVERCEL™ NON-ADHERENT Dressing is used to help manage wound fluid and protect from infection in moderate to highly draining shallow and deep wounds such as pressure ulcers, diabetic foot ulcers and leg ulcers.

How to apply


Preparing the wound

Prepare the wound according to appropriate wound management protocol as instructed by your clinician

Dressing selection


Select a size of SILVERCEL™ NON-ADHERENT Dressing that is slightly larger than the wound as instructed by your clinician

Dressing preparation

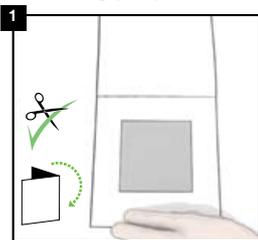


1. Cut (using clean scissors) SILVERCEL™ NON-ADHERENT Dressing or fold the dressing to fit the wound.
2. Ensure SILVERCEL™ NON-ADHERENT Dressing does not overlap the wound edges onto the healthy skin.

Dressing application

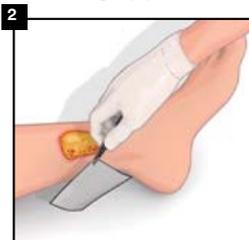


1. Gently apply SILVERCEL™ NON-ADHERENT Dressing to the wound.
2. Loosely pack deep wounds. As wound conditions improve and exudate levels decrease, moisten the dressing with normal saline solution before application.

Secondary dressing application

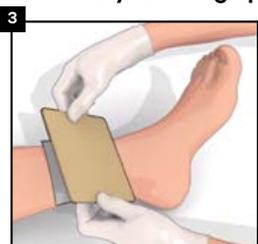

3. Cover SILVERCEL™ NON-ADHERENT Dressing with a secondary dressing such as a foam dressing and as instructed by your clinician.

Dressing change and removal

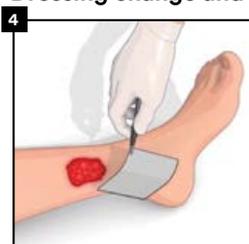

4. Gently remove the dressing from the wound bed and discard. If the wound appears dry, saturate the dressing with sterile saline solution prior to removal.

How often to apply

Dressing change frequency will depend on wound condition and the level of exudate. Reapply SILVERCEL™ NON-ADHERENT Dressing when the secondary dressing has reached its absorbent capacity or as recommended by your clinician.

NOTE: Specific indications, contraindications, warnings, precautions and safety information exist for these products and therapies. Please consult a clinician and product instructions for use prior to application. The instructions for use are intended for healthcare professionals.

NOTE: This document is intended for patient self-care as guided by their HCP and is not intended to replace the Instructions for Use.

©Copyright 2020 3M. All rights reserved. 3M and the other marks shown are marks and/or registered marks. Unauthorized use prohibited. PRA-PM-ALL-00364 (04/20)

